

G-Med™

Safety Scalp Vein Set for Single Use

GALAXY-MED™
Medical Safe Products

mobile +48 601 250 088

www.g-med.pl

e-mail: office@g-med.pl

Intended Use:

The G-Med™ Safety Scalp Vein Set with safety feature (for single use) is intended for venepuncture and used for intravenous injection of fluids or medication into a patient or to sample blood. It is designed with a safety sheath which is manually activated over the needle following removal from the patient. The needle is fully enclosed within the sheath when activated to prevent the risk of needlestick injury.

Specifications:

19G, 20G, 21G, 22G, 23G, 24G, 25G, 26G, 27G



SIZE		COLOR
OD (MM)	GAUGE	
0,40	27G	Gray
0,45	26G	Brown
0,50	25G	Orange
0,55	24G	Purple
0,60	23G	Blue
0,70	22G	Black
0,80	21G	Green
0,90	20G	Yellow
1,10	19G	Cream

Product Features:


- Butterfly Type.
- Soft wing for easy grip and application.
- Unique protective sheath designed to protect from needlestick injury.
- ISO needle gauge colour coded wing for easy recognition.
- Luer Lock and Luer Slip fitting.
- Sharp needle to maximize patient comfort.

Safety Feature:

After use, the needle is retracted into the Safety Sheath by gripping the base of the sheath and then pulling on the tube. The sheath will cover the needle avoiding the risk of needlestick.


Once the needle is completely contained and locked inside the sheath, it cannot be used again.

Instructions For Use:

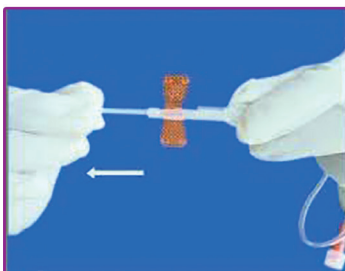


Remove winged set from packaging and ensure puncture area is disinfected according to standard protocol.

Remove Needle Sheath.

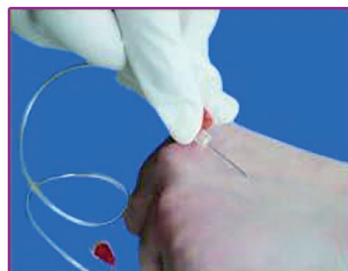

Using thumb and index finger, grasp the wings and insert the needle into the vein following standard technique.

A blood 'flash' will indicate that the needle is in the vein.

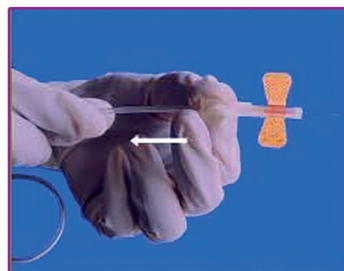

After infusion remove the needle from the patient following standard protocol.

Commence activation of the safety mechanism by gripping the base of the safety sheath with the thumb and forefinger.



Pull gently on the tubing and slide the safety sheath over the needle.

Ensure the needle is locked securely.

Dispose of the scalp vein set in an approved sharps container.

G-Med™

Safety Scalp Vein Set for Single Use

GALAXY-MED™
Medical Safe Products

mobile +48 601 250 088

www.g-med.pl

e-mail: office@g-med.pl

Packaging Dimensions:

Product	Size	Blister	Box (cm)/pcs	Carton (cm)/pcs
Safety Scalp Vein Set	All sizes	12.5 x 6.5cm	23 x 12x 13 cm / 100pcs	62 x 48x 28.5 cm / 2000 pcs

Product Code	Needle Gauge	Needle Length	Wing Colour	Tube Length 7"	Tube Length 12"
1601 - 7	19 G (1.10 mm)	¾" (19 mm)	Cream	✓	
1602 - 7	20 G (0.90 mm)	¾" (19 mm)	Yellow	✓	
1603 - 7	21 G (0.80 mm)	¾" (19 mm)	Green	✓	
1604 - 7	22 G (0.70 mm)	¾" (19 mm)	Black	✓	
1605 - 7	23 G (0.60 mm)	¾" (19 mm)	Blue	✓	
1606 - 7	24 G (0.55 mm)	¾" (19 mm)	Purple	✓	
1607 - 7	25 G (0.50 mm)	¾" (19 mm)	Orange	✓	
1608 - 7	26 G (0.45 mm)	¾" (19 mm)	Brown	✓	
1609 - 7	27 G (0.40 mm)	¾" (19 mm)	Grey	✓	
1601 - 12	19 G (1.10 mm)	¾" (19 mm)	Cream		✓
1602 - 12	20 G (0.90 mm)	¾" (19 mm)	Yellow		✓
1603 - 12	21 G (0.80 mm)	¾" (19 mm)	Green		✓
1604 - 12	22 G (0.70 mm)	¾" (19 mm)	Black		✓
1605 - 12	23 G (0.60 mm)	¾" (19 mm)	Blue		✓
1606 - 12	24 G (0.55 mm)	¾" (19 mm)	Purple		✓
1607 - 12	25 G (0.50 mm)	¾" (19 mm)	Orange		✓
1608 - 12	26 G (0.45 mm)	¾" (19 mm)	Brown		✓
1609 - 12	27 G (0.40 mm)	¾" (19 mm)	Grey		✓

Specifications:

Cannula	: Stainless Steel
Tube Length	: 7 inch (177.8 mm) or 12 inch (300 mm)
Connector	: Luer Lock or Luer Slip
Wing	: Colour coded to ISO needle gauge size colour coding
Sterilization	: ETO

GALAXY-MED™
Medical Safe Products

mobile +48 601 250 088

www.g-med.pl • e-mail: office@g-med.pl